

# DENTO-VIRACTIS 57

## Cleaning and disinfection of rotary instruments and cutters

Pour DENTO-VIRACTIS 57 directly in the burs tray

STEP 1



Immerse burs and let soak for 15 minutes

STEP 2



Brush in order to eliminate the residue if needed

STEP 3



Rinse the instruments best with distilled water or tap water

STEP 4



Dry each instrument with a clean paper towel

STEP 5



Pack the instruments in a sterilisation pouch, then sterilise

STEP 6



### ADVANTAGES

- Adapted for all types of materials: steel, tungsten and diamond burs
- Kills bacteria, viruses, moistures and biofilm
- Alkaline pH to ensure the quick elimination of infectious agents
- Ready to use product
- Product dedicated to rotary instruments and burs for higher disinfecting properties

### CAUTION

- Renew the cleaning bath every day
- Product for external use only. Do not ingest
- Precautions for use: refer to the medical safety data sheet and the label..

Product classified according to the European Directive "dangerous preparations" 1999/45/EC.

### MICROBIOLOGICAL PROPERTIES

- **Bactericidal:** EN 1040, EN 13727, EN 14561, NF T 72-190, NF T 72-301
- **Fungicidal:** EN 1275, EN 13624, EN 14562 (Candida Albicans)
- **Virucidal efficacy:** PRV (surrogate of Hepatitis B), Herpes virus simplex type 1, BVDV (surrogate of HCV), Enterovirus polio 1, Adenovirus type 5, Orthopoxvirus of the vaccine

DV57-PEN-V1