3M ESPE

Express XT Putty Quick

Express XT Putty Soft

VPS Impression Material

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3M ESPE AG
Dental Products
D-82229 Seefeld - Germany

3M ESPE
Dental Products
St. Paul, MN 55144-0000

3M ESPE, Express, Garant and Impresst are trademarks of 3M or 3M ESPE AG.
© 3M 2007. All rights reserved.

ENGLISH

Product Description
Express XT Impression materials, manufactured for 3M ESPE, are addition type silicones for one-step and two-step impression technique. Express XT Putty Quick and Express XT Putty Soft are kneadable materials for hand-mixing. All other materials are available in the Garant cartridge and are statically mixed and directly applied with the Garant Dispenser, manufactured for 3M ESPE.

Product Name	ISO 4823	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Express XT Putty Quick	Type 0	putty	green	1:1	manually
Express XT Putty Soft	Type 0	putty	ocher	1:1	manually
Express XT Light Body Quick	Type 3	light-bodied	pink	1:1	Garant Dispenser
Express XT Light Body	Type 3	light-bodied	magenta	1:1	Garant Dispenser
Express XT Regular Body Quick	Type 2	medium-bodied	caramel	1:1	Garant Dispenser
Express XT Regular Body	Type 2	medium-bodied	petrol	1:1	Garant Dispenser

These Instructions for Use should be kept for the duration of product use. Additionally mentioned products please refer to the corresponding Instructions for Use.

Areas of Application
All precision impressions (of e.g.: crown, bridge, inlay and onlay preparations).

Precautionary Measures
3M ESPE MSDSs can be obtained from www.mm.com or contact your local subsidiary.

Preparation
► Insert undercut or interdental spaces (as well as any other impression material with increased shore hardness) sufficiently to aid impression removal from the teeth after setting, as otherwise impression removal from the mouth will be difficult or could lead to the extraction of natural teeth or prostheses.

Impression Tray:
Suitable impression trays generally used for precision impressions are suitable.
► For sufficient adhesion, apply VPS Tray Adhesive, manufactured by 3M ESPE, on the tray and allow it to dry completely, 5-15 minutes drying time are ideal.

Garant Dispenser:
► Insert the cartridge with the wash impression material into the Garant Dispenser.
► Prior to application, check that both Garant cartridge openings are not clogged.

► When using new Garant cartridges, extrude a small amount of paste and catalyst pastes into a cup until base paste and catalyst are evenly extruded.
► Attach a yellow Garant Mixing Tip, manufactured for 3M ESPE, to the cartridge.
► For intraoral application attach a yellow Garant Intraoral Tip onto the Garant Mixing Tip.
► If required, enlarge the exit opening for the intraoral tip by cutting it with a scalpel.

► Make sure that base paste and catalyst are mixed completely and are ready for use.
- Paste color: see "Product Description".
- Material that has set inside the Garant Mixing Tip should not be extruded by force as this may damage the Garant cartridge and Garant Mixing Tip and result in the formation of leaks.

Retraction
Suitable retraction agents include aluminum hydroxide chloride, aluminum sulfate or ferric sulfate solutions.
► Keep the areas from which the impression is to be taken dry during drying impression.
► The use of retraction cords may be required for subgingival preparations.
► Prior to the impression, thoroughly remove residual retraction material by rinsing and drying.

Dosage and Mixing of the Putty Material
► After opening a new container, remove and discard the foil.
► Use the pertinent dosage spoon to take equal volumes of the base paste and catalyst pastes. Express XT Putty Quick and Express XT Putty Soft require the dosage spoons must be filled level. The total volume depends on the impression tray size.
► Knead the pastes manually for 30 seconds. The mixed paste must display an even color without stripping.
- Paste color: please see "Product Description".

Product	Working Time at 23°C/74°F		Intraoral Setting Time	Intraoral Setting Time
	min:sec	min:sec		
Express XT Putty Quick	1:30	-	2:30	-
Express XT Light Body Quick	1:30	0:40	2:30	-
Express XT Regular Body Quick	1:30	0:40	2:30	-
Intraoral setting time for combinations of quick setting materials and one-step technique: 2:30				
Express XT Putty Soft	2:00	-	3:30	-
Express XT Light Body	2:00	1:00	3:30	-
Express XT Regular Body	2:00	1:00	3:30	-
Intraoral setting time for combinations of regular setting materials and one-step technique: 3:30				

* Start of mixing = Entry of paste into the mixing tip.
These are the processing times for products that have been kept at a temperature of 23°C/74°F and 20-80% relative humidity. These times are shortened due to higher temperatures of the product or prolonged if the temperature of the product is lower.

Impression Taking

Recommended product combinations	
Express XT Putty Soft	Express XT Light Body or Express XT Regular Body
Express XT Putty Quick	Express XT Light Body Quick or Express XT Regular Body Quick

- Fill Express XT Putty Quick or Putty Soft into the impression tray prepared with adhesive.
- While or after the tray is being loaded, apply the wash material around the cleaned and dried preparation from the bottom up. Keep the application nozzle permanently immersed in the paste to avoid the formation of air bubbles.
 - Do not exceed tray material and wash material processing times to avoid incomplete or inaccurate impressions.
- Slightly place the loaded tray in the mouth parallel to the long axes of the prepared teeth, and hold it in position passively.
- Remove the impression from the mouth after completed setting.

Two-step Technique:	
Recommended product combinations	
Express XT Putty Soft or Express XT Putty Quick	Express XT Light Body or Express XT Light Body Quick

- Fill Express XT Putty Quick or Putty Soft into the impression tray prepared with adhesive.
- Slightly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth, and hold it in place without exerting pressure.
- Remove the impression from the mouth after completed setting, cleaned and dried.
- Excise undercut and interdental septa etc. and cut spillways, if required.
- Clean and dry the impression thoroughly to provide optimal coalescence of the two materials.
- Insert the wash impression material into the preliminary impression at the appropriate sites. If required, apply nozzle around the cleaned and dried preparation, while keeping the application nozzle permanently immersed in the paste to avoid the formation of bubbles.
- Replace the preliminary impression in the mouth and hold it in place.
- Remove the impression from the mouth after completed setting.

After Impression Taking
Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene
► Place the impression in a standard disinfectant solution, e.g.: Impresst™, manufactured for 3M ESPE. The duration of contact is according to the instructions of the manufacturer. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
► Clean the impression in a standard disinfectant solution by rinsing under running water for approx. 15 remaining. Impresst is not available in all countries.

Model Preparation
► The impression can be poured two hours after removal from the mouth.
► There are no other time limitations for pouring.
► In order to attain a bubble-free model, rinse the impression with water shortly before pouring, remove excess water or use a silicone surfactant.
► Pour the impression with a standard dental die or model stone. The impressions can be silver or copper plated.

Cleaning
► Garant Dispenser: Remove paste that has not set using an alcohol-soaked swab. The dispenser handle and plunger can be autoclaved up to a temperature of 135°C/275°F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
► Impression Tray: The adhesive can be removed from re-usable impression trays using acetone.

Notes
► Completely remove the oxygen inhibition layer of composite materials, e.g.: fillings or core buildsups, it may impair or retard the setting of silicone impression materials.

Keep the filled Garant Mixing Tip on top of the cartridge to keep a closure until the next use. Removing and replacing the spent Garant Mixing Tip will lead to carry-over of paste and ensuing formation of clots.

- Vinyl polysiloxane impression materials must not be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

Storage and Stability
Store the product at 15-25°C/59-77°F.
Do not use after the expiration date.
Store set impressions at temperatures below 30°C/86°F.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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