## W&H Service Oil F1, MD-400

<b>D</b>	<b>2</b>	<b>Gebrauchsanweisung</b>	<b>PL</b>	<b>24</b>	<b>Instrukcja obsługi</b>
<b>GB</b>	<b>4</b>	<b>Instructions for use</b>	<b>RU</b>	<b>26</b>	<b>Инструкция по эксплуатации для серии</b>
<b>F</b>	<b>6</b>	<b>Notice d'utilisation</b>	<b>CZ</b>	<b>28</b>	<b>Návod k použití</b>
<b>NL</b>	<b>8</b>	<b>Gebruiksaanwijzing</b>	<b>TR</b>	<b>30</b>	<b>Kullanma talimatı</b>
<b>E</b>	<b>10</b>	<b>Instrucciones de uso</b>	<b>H</b>	<b>32</b>	<b>Használati utasítás</b>
<b>P</b>	<b>12</b>	<b>Instruções de Utilização</b>	<b>SLO</b>	<b>34</b>	<b>Navodila za uporabo</b>
<b>I</b>	<b>14</b>	<b>Istruzioni di impiego</b>	<b>HR</b>	<b>36</b>	<b>Upute za uporabu</b>
<b>S</b>	<b>16</b>	<b>Bruksanvisning</b>	<b>SK</b>	<b>38</b>	<b>Návod na používanie</b>
<b>SF</b>	<b>18</b>	<b>Käyttöohje</b>	<b>BG</b>	<b>40</b>	<b>Упътване за употреба</b>
<b>DK</b>	<b>20</b>	<b>Brugsanvisning</b>	<b>RO</b>	<b>42</b>	<b>Instrucțiuni de utilizare</b>
<b>GR</b>	<b>22</b>	<b>Οδηγίες Χρήσεως</b>	<b>RC</b>	<b>44</b>	<b>使用说明</b>

## Ölpflege mit W&H Service Oil F1, MD-400

---

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---

### Intended use

The W&H Service Oil F1 is intended for the lubrication of dental transmission instruments, turbines, air motors and air-driven calculus removal instruments.

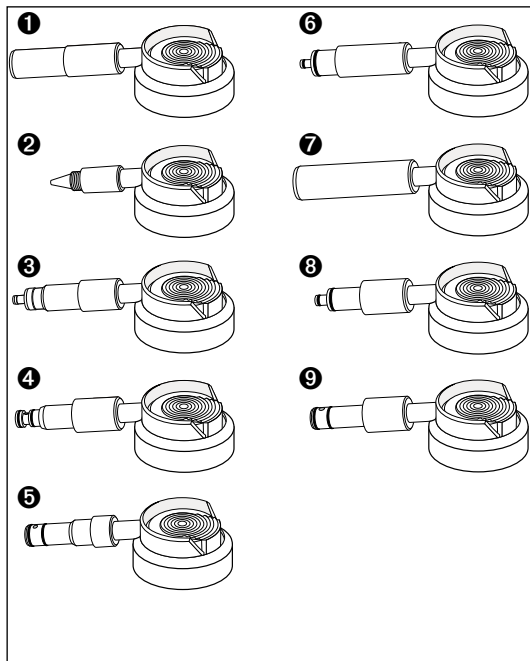
Comply with the safety instructions on the can!

- > Fit the spray cap with the appropriate nozzle onto the spray can.
- > Shake the spray can before each application.
- > Insert the spray nozzle into the instrument.
- > Hold the instrument firmly.
- > Hold the can upright and spray for about 1 second.
- > Follow the instructions for lubrication of W&H instruments in the relevant instructions for use.

Keep the spray caps with nozzles and use them with the next oil spray can you purchase!

## Oil service with W&H Service Oil F1, MD-400

### Spray caps with nozzles for

- 1 straight and contra-angle handpieces REF 02038200
- 2 chuck system for turbines and contra-angle handpieces as well as turbine handpieces and air motors with fixed connection REF 02036100
- 3 W&H turbines, air motors and air scalers with Roto Quick connection REF 02229200
- 4 Multiflex® connection REF 04396100
- 5 A-dec LW connection REF 03522500
- 6 Bien Air Unifix® connection REF 05772000
- 7 NSK Mach® connection REF 05773300
- 8 NSK QD-J® connection REF 05907800
- 9 Sirona® connection REF 05908000

*Multiflex® is a registered trade mark of Kaltenbach & Voigt GmbH & Co.KG, Biberach  
BienAir Unifix® is a registered trade mark of Bien-Air Dental SA, Bienne  
NSK Mach® and NSK QD-J® is a registered trade mark of NSK NAKANASHI INC., Japan  
Sirona® is a registered trade mark of Sirona Dental Systems GmbH, Bensheim*

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**W&H Dentalwerk Bürmoos GmbH**